

Nuclear Green Solution

REF CS-0004-20

Σ 40 (20 ml)

For use in *in situ* hybridization procedures

CE

IVD


In vitro diagnostic medical device

according to EU directive 98/79/EC



Nuclear Green Solution

REF CS-0004-20

 40 (20 ml)

For use in *in situ* hybridization procedures

CE

IVD

In vitro diagnostic medical device

according to EU directive 98/79/EC

1. Scope of Application

This product is designed for *in vitro* diagnostic use (according to EU directive 98/79/EC). Interpretation of results must be made within the context of the patient's clinical history with respect to further clinical and pathologic data of the patient by a qualified pathologist!

The Nuclear Green Solution (**CS4**) is designed to be used for counterstaining steps in *in situ* hybridization (ISH) protocols of ZytoVision.

2. Safety Precautions and Disposal

- ✓ Read the operating instructions prior to use!
- ✓ Do not use the reagents after the expiry date has been reached!
- ✓ Avoid any cross-contamination and micro-bacterial contamination of the reagents!
- ✓ Avoid any direct contact with the reagents. Take appropriate protective measures (use disposable gloves, protective glasses, and lab garments)!
- ✓ If reagents come into contact with skin, rinse skin immediately with copious quantities of water!
- ✓ Never pipet solutions with your mouth!
- ✓ The disposal of reagents must be carried out in accordance with local regulations!
- ✓ A material safety data sheet is available on request for the professional user!

3. Package Content

Code	Component	Quantity	Container
CS4	<u>Nuclear Green Solution</u>	20 ml	Screw-cap bottle, black
	Instruction manual	1	

Component (**CS4**) is sufficient for 40 reactions.

4. Storage and Shelf Life

The component must be stored at 2...8°C. If these storage conditions are followed, the component will function, without loss of performance, at least until the expiry date printed on the label.

5. Instructions

Nuclear Green Solution (CS4) is a ready-to-use solution for counterstaining steps in ISH applications, following the ISH protocols of ZytoVision.

Our experts are available to answer your questions.


As of: August 1, 2017 (5.0)

Trademarks:

ZytoVision® is a trademark of ZytoVision GmbH.



ZytoVision GmbH · Fischkai 1
D - 27572 Bremerhaven · Germany
Phone: +49 (0) 471/4832 - 300
Fax: +49 (0) 471/4832 - 509
www.zytovision.com
info@zytovision.com

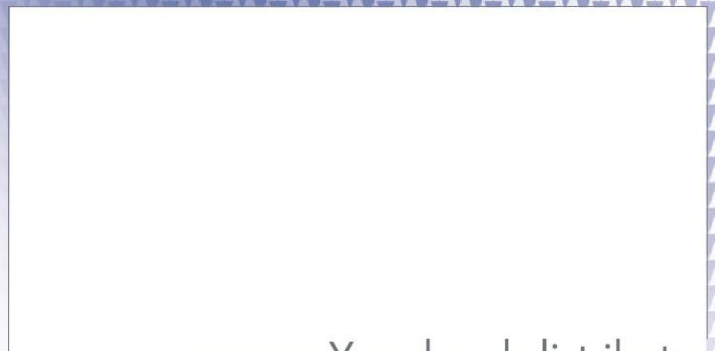

Your local distributor